

Important Safety Information for
AIR OPTIX® Family of Contact Lenses

Important information for AIR OPTIX® NIGHT & DAY® AQUA (lotrafilcon A) contact lenses:

AIR OPTIX® NIGHT & DAY® AQUA lenses are indicated for vision correction for daily wear (worn only while awake) or extended wear (worn while awake and asleep) for up to 30 nights. Relevant Warnings: A corneal ulcer may develop rapidly and cause eye pain, redness or blurry vision as it progresses. If left untreated, a scar, and in rare cases loss of vision, may result. The risk of serious problems is greater for extended wear vs. daily wear and smoking increases this risk. A one-year post-market study found 0.18% (18 out of 10,000) of wearers developed a severe corneal infection, with 0.04% (4 out of 10,000) of wearers experiencing a permanent reduction in vision by two or more rows of letters on an eye chart. Relevant Precautions: Not everyone can wear for 30 nights. Approximately 80% of wearers can wear the lenses for extended wear. About two-thirds of wearers achieve the full 30 nights continuous wear. Side Effects: In clinical trials, approximately 3-5% of wearers experience at least one episode of infiltrative keratitis, a localized inflammation of the cornea which may be accompanied by mild to severe pain and may require the use of antibiotic eye drops for up to one week. Other less serious side effects were conjunctivitis, lid irritation or lens discomfort including dryness, mild burning or stinging. Contraindications: Contact lenses should not be worn if you have: eye infection or inflammation (redness and/or swelling); eye disease, injury or dryness that interferes with contact lens wear; systemic disease that may be affected by or impact lens wear; certain allergic conditions or using certain medications (ex. some eye medications). Additional Information: Lenses should be replaced every month. If removed before then, lenses should be cleaned and disinfected before wearing again. Always follow the eye care professional's recommended lens wear, care and replacement schedule. If you have questions, ask your eye care professional or contact Alcon at (800)-875-3001 or airoptix.com.

Package Insert for AIR OPTIX® NIGHT & DAY® AQUA (lotrafilcon A) contact lenses

Patient Instruction Booklet for AIR OPTIX® NIGHT & DAY® AQUA (lotrafilcon A) contact lenses

Important Information for AIR OPTIX® AQUA, AIR OPTIX® for ASTIGMATISM, and AIR OPTIX® AQUA MULTIFOCAL (lotrafilcon B), contact lenses:

For daily wear or up to 6 nights extended wear for near/far-sightedness, astigmatism and/or presbyopia. Risk of serious eye problems (i.e., corneal ulcer) is greater for extended wear. In rare cases, loss of vision may result. Side effects like discomfort, mild burning or stinging may occur.

Package Insert for AIR OPTIX® AQUA (lotrafilcon B), AIR OPTIX® for Astigmatism (lotrafilcon B) and AIR OPTIX®AQUA Multifocal (lotrafilcon B) contact lenses

Patient Instruction Booklet for AIR OPTIX® AQUA (lotrafilcon B), AIR OPTIX® for Astigmatism (lotrafilcon B) and AIR OPTIX®AQUA Multifocal (lotrafilcon B) contact lenses

For more information, call (800) 875-3001 or visit our Alcon website.

AIR OPTIX, NIGHT & DAY, OPTI-FREE, PureMoist, and the AIR OPTIX logo are trademarks of Novartis AG.

Alcon, a Novartis AG Company

Important Information for AIR OPTIX® COLORS (lotrafilcon B) contact lenses:

For daily wear only for near/far-sightedness. Contact lenses, even if worn for cosmetic reasons, are prescription medical devices that must only be worn under the prescription, direction and supervision of an eye care professional. Serious eye health problems may occur as a result of sharing contact lenses. Although rare, serious eye problems can develop while wearing contact lenses. Side effects like discomfort, mild burning or stinging may occur. To help avoid these problems, follow the wear and replacement schedule and the lens care instructions provided by your eye doctor.

Package Insert for AIR OPTIX® COLORS contact lenses

Patient Instruction Booklet for AIR OPTIX® COLORS contact lenses